

# gb SARS-CoV-2 Multiplex EndoC

Cat. no.: 3234-100  
3234-500  
3234-M



generi biotech

## PURPOSE OF USE

gb SARS-CoV-2 Multiplex EndoC enables the detection of SARS-CoV-2 virus (Wuhan coronavirus 2019). Detection is based on the viral E and RdRP genes. The assessment originates from protocol published by Charité laboratory (Berlin) and Institut Pasteur laboratory (Paris). False negativity is checked by endogenous control that detects human gene GUSB.

## PRINCIPLE OF THE TEST

The test is based on a **one-step RT-qPCR** methodology. The kit contains all the necessary components to perform the test.

## INSTRUCTIONS FOR USE

- 1) Let the reagents thaw completely, mix them thoroughly and spin down briefly prior each use.

### Performing RNA isolation

- 2) RNA Isolation does not require any additional steps. Perform the RNA isolation according to your standard protocol.

### Performing RT-qPCR


- 3) Pipette **10 µl** (× number of samples) of **Master Mix OneStep Multi** (tube with a blue cap) and **5 µl** (× number of samples) of **Assay CoV-2 Multi EC** (tube with a green cap) into the new tube (not included), mix well, spin down, and mark the date of preparation. In this way a **complete assay** is prepared. Handle the mixture according to the instructions in the chapter Storage and manipulation conditions.
- 4) Dispense the prepared complete assay by **15 µl** into micro tubes or plate wells.
- 5) Add **5 µl** of template to the assay and spin briefly.
  - For each run, an analysis of **Positive Control CoV-2 EC** and **Deionized Water** as the NTC control is required for proper evaluation.
  - Analyse RNA as soon as possible after isolation.
- 6) Perform sample analysis immediately after reaction mixture preparation.

## Amplification protocol and data collection

- Set the PCR cycler to the following temperature profile:

Reverse transcription	42 °C	10 min	45 cycles
Initial denaturation	95 °C	3 min	
Denaturation	95 °C	10 sec	
Annealing + Elongation (+ fluorescence acquisition)	60 °C	30 sec	

- The total volume of a PCR reaction is **20 µl**; please consider this fact when setting up the cycler.
- When using the Rotor-Gene instrument, **Gain** needs to be set for all channels, so that the basic fluorescence is within **5–10 RFU**. **Attention** – in case of setting the Gain before the analysis, the temperature of Gain optimisation **must not exceed 42 °C**.
- Fluorescence acquisition must be set active for the FAM/SYBR, HEX/JOE/VIC, and Cy5 channels.
- Instructions for setting up a cycler can be found at: <https://www.generi-biotech.com> – see the “download” section, or follow the instrument manufacturer’s instructions for use.

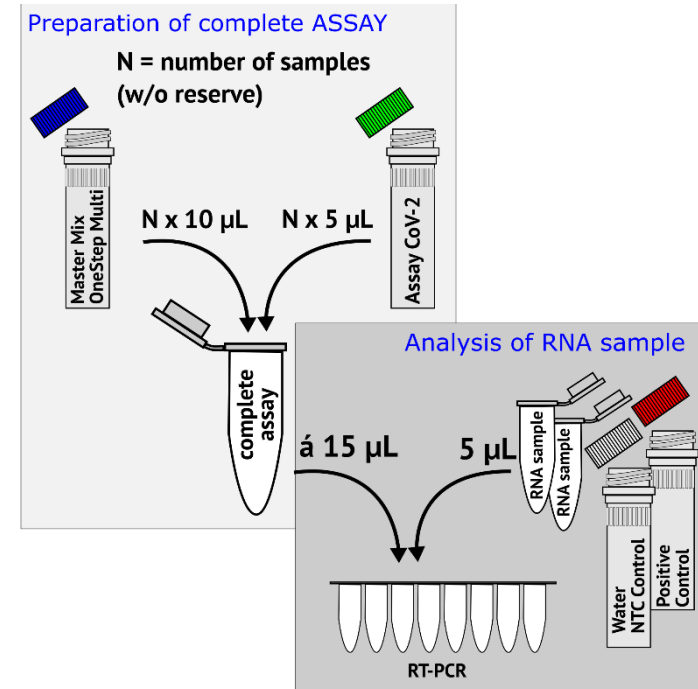

## DATA ANALYSIS

Determine the SARS-CoV-2 presence in a sample by analysing Ct values obtained with the software of your real-time PCR cycler. The limit of detection (LOD) for this kit is 3 copies/rxn. LOD has been set at a 95% confidence interval.

Read the signal as Ct (threshold cycle) in a given fluorescence channel in the quantitation mode. Set the **threshold fluorescence value (threshold)** at approximately 10% level of fluorescence of positive control in last cycle. Read Ct values **for viral genes in FAM and HEX channels, Ct values for human endogenous control in the Cy5 channel**. For strongly positive samples, the human endogenous control may not be amplified. Follow the instructions of the cycler manufacturer.

First verify the analysis validity. Display signals of control samples.

### Analysis validity

Analysis is considered valid when control signals correspond with the following layout:

- Positive Control CoV-2 EC – signals in the FAM/SYBR, HEX/JOE/VIC, and Cy5 channels
- Deionized Water as negative isolation control or NTC – no signal

If the analysis is valid, continue with evaluation of the samples. Otherwise follow recommendations given in the Troubleshooting chapter.

## Interpretation of results

The final outcome of the analysis is the evaluation of the SARS-CoV-2 presence (i.e., sample positivity).

VALID results	FAM	HEX	Cy5
Positive	+	+	+/-
Negative	-	-	+
Low positivity *	Ct ≥ 35	-	+
	-	Ct ≥ 35	+
Positive for other virus than SARS-CoV-2 from Sarbecovirus subgenus **	Ct < 35	-	+/-

INVALID results	FAM	HEX	Cy5
Incorrect sampling, failed isolation or inhibition of RT-PCR occurred, repeat analysis	-	-	-
	Ct ≥ 35	-	-
Unreliable result **	-	Ct ≥ 35	-
	-	Ct < 35	+/-

\* At samples with low positivity the signal can be acquired only in one of the channels FAM/HEX but the endogenous control in Cy5 channel must be detected in such cases, otherwise the result is invalid

\*\* For these results, we recommend an independent analysis with the **gb** Sarbeco N (primary test) and **gb** SARS-CoV-2 N (confirmation test) kits to verify the presence of virus based on the N gene.

## TROUBLESHOOTING

Test results can be considered correct only if the instructions indicated in the enclosed manual are followed. Where control samples give incorrect results, check the following:

- the expiry date of the kit
- the storage and manipulation conditions
- the pipette and cycler settings

Finding:	Corrective action suggestion:
A FAM and/or HEX signal is detected in the negative control (Deionized Water) reaction.	Reactions were most probably contaminated with a template. Repeat the analysis.
In the negative control reaction, a FAM and/or HEX signal is repeatedly detected by the same detection assay.	The assay was most probably contaminated with a template. Repeat the analysis with a new aliquot of the assay.
In the negative control reaction, a FAM and/or HEX signal is repeatedly detected by all the detection assays.	Most probably, the component Deionized Water was contaminated with a template. Repeat analysis with a new aliquot of the water of PCR quality.
Standard Positive Control is not detected or is detected only in one/two of the channels.	A pipetting error probably occurred. Repeat the analysis.
A signal in the FAM and/or HEX channel is detected in the negative isolation control; however, no signal is detected in the PCR negative control (NTC).	A cross-contamination between samples probably occurred. Prepare new RNA isolates.
No signal in any channel is detected for the examined sample, though the analysis was evaluated as valid.	The inhibition of PCR probably caused a failure of the analysis. Perform an analysis with a new RNA isolate.

## CONTENTS AND DESCRIPTION OF KIT COMPONENTS

Component <sup>1)</sup>	Volume	Qty <sup>2)</sup>	Concentration
● Assay CoV-2 Multi EC	0.5 ml <sup>3)</sup>	1   5   10	4×
● Master Mix OneStep Multi	1.0 ml <sup>3)</sup>	1   5   10	2×
● Positive Control CoV-2 EC	0.2 ml	1   1   2	4×
○ Deionized Water	1.0 ml	1   1   2	

<sup>1)</sup> Tube lid colour corresponds with reagent type.

<sup>2)</sup> Number for kit size of 100 | 500 | 1000 reactions.

<sup>3)</sup> Volume equates to 100 PCR reactions of 20 µl of volume.

### Assay CoV-2 Multi EC

Assay CoV-2 Multi EC is a mixture of amplification primers and fluorescently labelled probes. The probes allow the detection of **viral gene E in the FAM channel** ( $\lambda_{\text{EXCITATION}} = 495 \text{ nm}$ ,  $\lambda_{\text{EMISSION}} = 520 \text{ nm}$ ); **viral gene RdRP in the HEX channel** ( $\lambda_{\text{EXCITATION}} = 535 \text{ nm}$ ,  $\lambda_{\text{EMISSION}} = 556 \text{ nm}$ ); and **human endogenous control in the Cy5 channel** ( $\lambda_{\text{EXCITATION}} = 650 \text{ nm}$ ,  $\lambda_{\text{EMISSION}} = 670 \text{ nm}$ ). We can thus detect up to three signals when the viral genes are present, whereas in their absence only the signal of human endogenous control (GUSB gene) in the Cy5 channel is detected. The Assay is supplied in a micro tube with a green cap. Mixing with Master Mix OneStep Multi provides a complete ready-to-use assay.

### Master Mix OneStep Multi

Master Mix OneStep in the blue cap micro tube is an optimized mixture of buffer, reverse transcriptase, polymerase and nucleotides that is necessary for RT-qPCR.

### Positive Control CoV-2 EC

Positive Control CoV-2 EC serves as a positive control (a standard) for a verification of the analysis validity. It is supplied in a micro tube with a red cap. Handle Positive Control to avoid cross-contamination with other kit components and analysed samples.

### Deionized Water

Deionized water serves as a negative isolation control and no-template control (NTC) in PCR. It is supplied in a tube with a transparent lid.

### Reagents and equipment not included in the kit

- kit or reagents for the isolation of viral RNA
- single-use plastic micro tubes, strips or plates convenient for use in a PCR cycler
- adjustable micropipettes with the corresponding range
- disposable pipette tips with filters
- laboratory vortex and centrifuge
- real-time PCR cycler with software

### WARNINGS AND PRECAUTIONS

#### Storage and manipulation conditions

- Store all kit components at a temperature below **-20 °C**.
- Assay is photosensitive; therefore, limit its handling in the light to the shortest time possible.
- Reagents are designed for work at laboratory temperature.
- Individual kit components may be repeatedly thawed and frozen **5 times at the most**. Do not freeze the complete assay resulting from mixing the Master Mix OneStep Multi and Assay CoV-2 E-RdRP. The final reaction mixture is **disposable**.
- If the above-mentioned conditions are followed, the kit is stable until its expiry date stated on the **box label**.

#### Safety measures

- The kit is designed for professional use only.
- When working with RT-qPCR reagents and material, always wear laboratory clothing and safety gloves.
- In case of skin or eye contact with reagents, rinse the affected area under running water.







### Instructions for use

- Always use the enclosed version of the manual. The corresponding version number is marked on the label inside the box.
- Inappropriate reagents handling or adjustments of the workflow may negatively influence results and thus it is necessary to strictly follow the pipetting volumes, incubation times and temperature conditions as stated in the manual.
- Adhere to the expiry date of the kit indicated on the box label.
- Do not combine components from different batches of the kit.
- If any of the kit components is damaged upon receipt, do not use it and contact the manufacturer immediately. Keep the component for the purposes of an eventual claim.
- Use calibrated pipettes and instruments.
- Dispose of all waste material in accordance with the applicable legislation. The outer packing is made from paper, the inner segment from polyurethane and the micro tubes from polypropylene. Reagents may be handled as common waste. Dispose of the final PCR analysis product taking into account the risk of work space contamination.

### Contamination precautions

- Assign specific spaces, equipment, material and protective equipment for the isolation of RNA/DNA from clinical material, and different ones for preparing RT-qPCR.
- Change your gloves and protective clothing whenever you suspect contamination.
- Never open an amplified PCR product in the place where the PCR reactions are prepared.
- Leave reagents open only for the time necessary to prepare PCR reactions.
- Use tips with filters when pipetting.
- When preparing a reaction mixture, take care not to contaminate any other component of the kit, or other samples, with the positive control. This may be avoided by closing all the micro tubes before manipulating a positive control.
- Use ultra-clean water for sample dilution; the Deionized Water provided with the kit may be used for this purpose.

## SYMBOLS USED ON STICKERS

	Batch code
	Expiry date
	Store at recommended temperature
	Contains
	Manufacturer
	Number of tests

## PRODUCT LINE

gb SARS-CoV-2 Multiplex	Cat. no. 3231
gb SARS-CoV-2 Combi	Cat. no. 3232
gb SARS-CoV-2 Influenza A/B	Cat. no. 3233
gb Sarbeco N (primary test)	Cat. no. 3229
gb SARS-CoV-2 N (confirmation test)	Cat. no. 3230

## REFERENCES

When using the kit, follow the manufacturer's manual for the cyclers. The list of cyclers on which the kit's performance parameters have been tested is available at the manufacturer's website.

For additional information please contact us at our e-mail address: [info@generi-biotech.com](mailto:info@generi-biotech.com) or by phone: +420 495 056 314. Further information can also be found on our website [www.generi-biotech.com](http://www.generi-biotech.com).



GENERI BIOTECH s.r.o.  
Machkova 587/42  
CZ-500 11, Hradec Kralove - Trebes  
CZECH REPUBLIC

[www.generi-biotech.com](http://www.generi-biotech.com)

Phone: +420 495 056 314

E-mail: [info@generi-biotech.com](mailto:info@generi-biotech.com)

**Version of the manual: 1.0**

Date of the last revision: 9. 11. 2020